


# Sterin-Trace

The Complete Sterilizer Record and Tracking System

## LABEL Print OPTIONS


(2 line label)

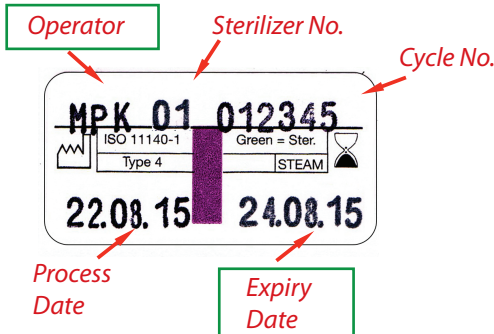

### Option 1

**Operator** may be identified using an allocated number or the initials of the person as show opposite.

**Expiry Date:** Most tracking systems have a provision for two dates on the label, as some Authorities require an expiry date to be displayed, usually 12 months.

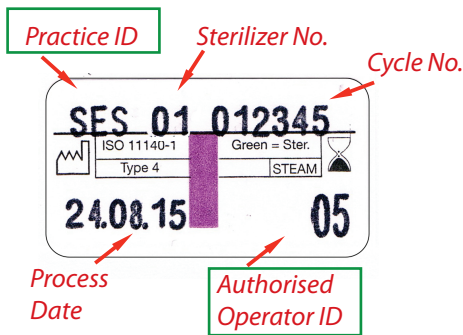

### Option 2

**Practice ID** or Department may be identified in this space rather than the Operator as show opposite.

**Authorised Operator ID** identifies the operator numerically and is recorded by the Practice Manager in the "Register of Authorised Operators" located at the back of the "Sterin-Trace" Steriliser Record Book.



## Setting the Print

Move the turning knob to the required position and turn it until the required character is in position. To function properly the white area of the printing position should be visible.

The first three (3) characters of the top line are selected following the table at the front of the printing head label. An example: If you want to print "A" then select "N" in the dialer area. To print "Q" you need to select "D" in the dialer area. For other characters please follow the table below:

A	B	C	D	E	F	G	H	I	J	K	L	M
N	O	P	Q	R	S	T	U	V	W	X	Y	Z

After setting all the characters in the right position for both lines one needs to secure the turning knob in its original position again. In this way the printing is secure.

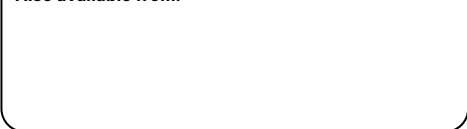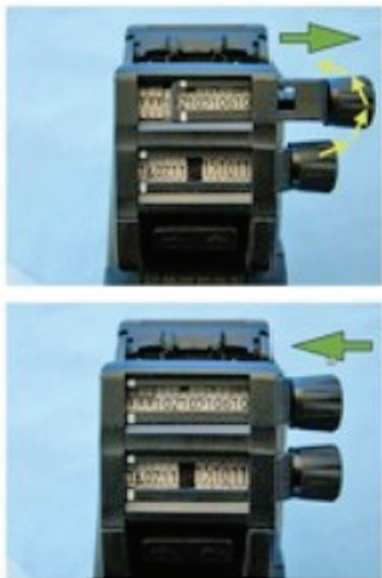



To be able to load labels into the label gun one should open the bottom of the label gun by pushing both levers at the side backwards. The bottom of the label gun will open itself after which one can push the bottom backwards to have a full access to place the label roll into the gun.



Unroll the label roll partly and **take of about 8 labels**. Push the roll into the label gun as per picture. The labels are pointing toward the top of the label gun.



After closing the bottom of the label gun one takes the backing of the label rolls and fold it backwards into the slot as per picture just behind the metal rolls. By operating the label gun handle a few times the backing material will be pulled in into the label gun. The label gun will align itself into the proper printing position.



In case of a rarely misprint one can adjust the printing position manually by loosen this screw and adjust the slide a bit to the back or front. After adjusting one should tight the screw again. The screw can be reached through the backing material.



Now the labels are placed the machine can be operated and labels will be printed and dispensed by the Sterintech 2-line label gun.

After a while the printing will become more grey and a change of the ink roller may be necessary.