




CAUTIONS !!!

HIBERNATED

DEVICE

According to common rules about worldwide shipment, all devices with batteries must travel with a low battery level.

To accomplish this rule, every STET CLEAN has been packed with a very low energy charge (<30%).

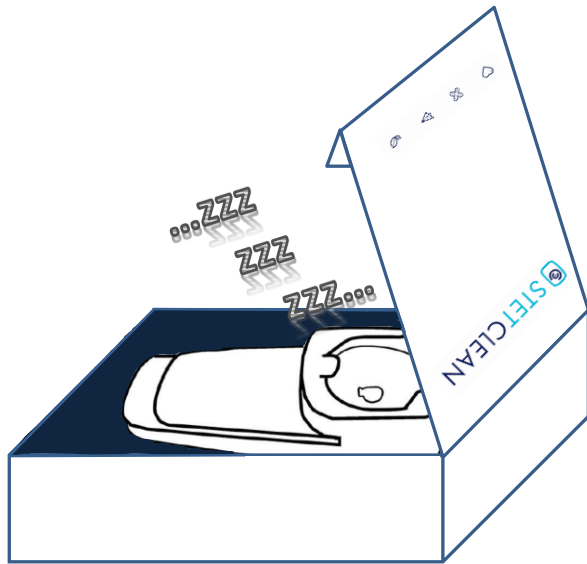

STET CLEAN software is designed to stay «hibernated» and avoid useless battery consumption, for shipping time and/or stocking phases (up to 1 year) because, as a matter of fact, long periods of no-recharge are a usually «danger» for batteries' quality and life.

Once you attach a Stethoscope or connect STET CLEAN to its USB cable, this hibernation stops and the device is awake.

If you do that, please remember that from that time on, you **HAVE TO RECHARGE THE DEVICE AT LEAST ONCE a WEEK**, otherwise the battery could lose its recharge capacity - even if it is in stand-by mode .

PLEASE READ THESE INFORMATIONS CAREFULLY