



Sterin-Trace

The Complete Sterilizer Record and Tracking System

The Sterin-Trace Sterilizer Record & Tracking System is designed to provide a simple method of record keeping & "Patient to Process" traceability of devices & instruments in the modern office based practice.

Sterin-Trace uses a simple intuitive check list that creates a reliable audit trail & enables the operator to record:

- | | |
|--|-----------------------------------|
| 1. Sterilizer cycles | 5. Registered Authorised Operator |
| 2. Process Expiry Date | 6. Patient |
| 3. Steam Penetration Test (Bowie-Dick) | 7. Maintenance Records |
| 4. Daily Test | |

"IF IT'S NOT RECORDED THEN IT IS NOT DONE"

- Record keeping is the evidence that the practice has performed & monitored all stages of the sterilization process.
- To comply with, AS/NZS 4187, AS/NZS 4815 & ISO 13683 the practice must maintain "Patient to Process" tracking of instruments & devices.
- Responsibility lies with the practice management to ensure all the cleaning & sterilization processes are performed & recorded.

The implementation of the Sterin-Trace Quality System within the practice will:

- Provide a legal reference should a challenge occur.
- Reduce the risk to patients.
- Minimize the chance of nonsterile items being used.




Marketed by:
Majac Medical Products Pty Ltd
 Tel: 1300 138 578 * Int: +617 3265 6355
 Email: sales@majacmedical.com.au
 www.majacmedical.com.au



**FS
540017**

Also available from:

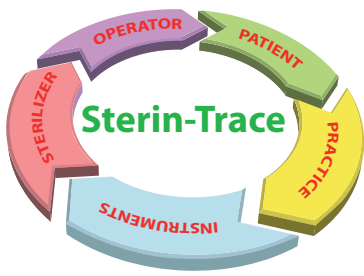

Sterin-Trace

The Complete Sterilizer Record and Tracking System

CYCLE BY CYCLE RECORD - BATCHING

Batching will link your sterile product from the process to the patient. This is easily achieved by using twin layer "piggy back" process labels. This process label is attached to the item or pack, then peeled off & placed onto the patient notes or card when used. An extra label should be processed & placed on the sterilizer record sheet.

Individual operators should have an ID number. Where a practice has multiple sterilizers then each sterilizer should have its own ID number which is linked to all daily validations.

THE STERIN-TRACE PROCESS LABEL

The Sterin-Trace Label acts as an indicator Type 4 (turns from Violet to Green), identifying items that have been through the steam sterilization process.

The Label will show: • Sterilizer Number • Cycle Number • Process Date • Expiry date • Operator



Operator
Sterilizer No.
Cycle No.
Process Date
Expiry Date
Type 4 Indicator

Sterin-Trace "Piggy Back" Process Label with Type 4 Steam Indicator

Code: 604.401.9000/12 : Box of 12 rolls (9000)

Code: 604.401.9000/3 : Box of 3 rolls (each 750)

Colours available: Blue, Red, Green, Yellow, Orange, Grey

Sharpie Industrial Marker Pen
Specially formulated ink for use on instruments & sterile packs
Code: S13801

PRINT OUTS - RETRIEVAL

There is no point keeping rolls of Autoclave "print out" streamers in a manner that prevents prompt retrieval. Using the Sterin-Trace System each sterilizer cycle "print out" is attached to the back page of the appropriate section of the Sterin-Trace Sterilizer Record System.



Sterin-Trace 2 Line Label Applicator

Code: 601.002.0001

Two lines ... but it carries all the information

The Sterintech two line label gun is reliable, easy to use and durable. It will serve for years of intensive usage. Sterintech double labels for two line label guns are available with all type of sterilization indicators as well as a variety of colors. Colored labels are used to assist FIFO systems as well as easy recognition of sterile goods for a certain department.

Often the combination of a double label gun label and normal tape is a more cost-effective solution than steam tapes only. Next to that they offer better readable data and prevent writing errors.

Characteristics:

- ◆ Process Indicators (Steam, FO, EtO) conform to ISO 11140 - Type 4
- ◆ Clear transition of indicator into its end-color
- ◆ Easy to use label gun
- ◆ Double labels serve for an easy traceability and documentation
- ◆ Non-Toxic / Lead Free indicators
- ◆ Latex free label materials
- ◆ Can be fully recycled

ST040417

Record keeping is the evidence that the practice has performed & monitored all stages of the sterilization process



Sterin-Trace

The Complete Sterilizer Record and Tracking System

Using a simple check list the Sterin-Trace System enables the operator to record:

- * Sterilizer
- * Steam Penetration Test (Bowie-Dick, Helix)
- * Authorised Operator
- * Maintenance Records
- * Calibration
- * Expiry Date

Tracking of instruments & devices from "Patient to Process"

Record keeping is the evidence that the practice has performed & monitored all stages of the sterilization process - if it is not recorded, it's not done.

Sterin-Trace Book will record 600 cycles & include a Sharpie marker pen which uses specially formulated ink for use on instruments & sterile packs.



Sharpie Marker Pen
S13801



Sterin-Trace Book
S602.RECORD

"If it is not recorded, it's not done"

Sterin-Trace "Piggy Back" Process Label with Type 4 Steam Indicator


2 Line Label Applicator
602.002.0001

604.401.9000/12 :
Box of 12 rolls (9000)


604.401.9000/3 :
Box of 3 rolls (each 750)

Sterintech 2 line label applicator is easy to use and reliable.

The Sterintech 2 line labels give a clear colour change to indicate steriliser performance. The 2 line label carries all information required for record keeping.

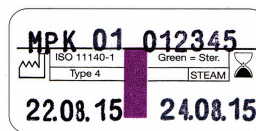

Changes from
Violet to Green

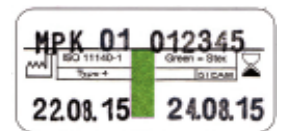

Record keeping is the evidence that the practice has performed & monitored all stages of the sterilization process

Sterintech™ Sterin-Trace 3 Line Label Applicator (Monarch Gun)

3 Line Label Steam - 12 rolls of 750 labels - Sterintech Fits Monarch Gun.

- 603.101.9000/12 Blue
- 603.102.9000/12 Red
- 603.103.9000/12 Green
- 603.104.9000/12 Yellow



3 Line Label steam - Plain (Grey) - most popular

- 603.211.9000/12 box of 12 rolls.
- 603.211.9000/3 pkt of 3 rolls (2250)

Available colours:
Blue, Red, Green, Yellow & Grey

Sterin-Trace 3 Line Applicator
603.003.0001

ST040417



MAJAC
HEALTHCARE INNOVATION

Marketed by:
Majac Medical Products Pty Ltd
Tel: 1300 138 578 * Int: +617 3265 6355
Email: sales@majacmedical.com.au
www.majacmedical.com.au



FS 540017

Also available from:

