



PLASTIC FREE DETERGENT WIPES

FEATURES

- Alcohol-free / disinfectant-free
- Suitable for commercial and household applications
- This product is not classified as hazardous or as dangerous goods.

QUANTITY

- Canister 100 wipes.
- Tub 400 wipes (refills 400 wipes)
- Softpack 80 wipes.

INGREDIENTS

Water, Sodium lauryl ether sulphate, Propoxy ethanol, Phenoxyethanol, Potassium sorbate, EDTA.

INSTRUCTIONS

Correct use of the CleanLIFE Medical Detergent Wipes helps remove organic matter prior to using CleanLIFE Medical Disinfectant or Isopropyl Alcohol Wipes. Use a new wipe for each new surface.

Five steps to effective medical cleaning:

- Pull open your tub, canister or softpack.
- Work from cleanest to dirtiest.
- Work from highest to lowest.
- Move in an S-shape pattern.
- Allow correct contact time.



DESCRIPTION

CleanLIFE Medical Detergent Wipes are ideal for cleaning dirt, dust, grime etc prior to disinfection. Suitable for commercial and household applications. The Formulated with a neutral pH. Leaves a streak-free finish.

SAFETY DIRECTIONS

Keep out of reach of children. Do not use as a personal wipe or on skin. Avoid contact with eyes and wash hands after use. Store in a cool dry place away from foodstuffs.



Marketed by:
Majac Medical Products Pty. Ltd.
Tel: 1300 138 578 • Int: +61 7 3265 6355
Email: sales@majacmedical.com.au
www.majacmedical.com.au



FS 540017