



# PLASTIC FREE ISOPROPYL ALCOHOL WIPES

## FEATURES

- Contains 70% Isopropyl Alcohol.
- Proven 99.99% efficacy against tested germs and bacteria from 60 seconds.
- Proven to kill the COVID-19 virus in 60 seconds.

## QUANTITY

- Canister 75 wipes and 150 wipes.
- Tub 400 wipes (refills 400 wipes).

## INGREDIENTS

Water, Isopropyl alcohol.

## INSTRUCTIONS

Correct use of the CleanLIFE Medical Isopropyl Alcohol Wipes helps remove organic matter and improves infection prevention and control. Use a new wipe for each new surface.

Five steps to effective medical cleaning:

- Pull open your tub, canister or softpack.
- Work from cleanest to dirtiest.
- Work from highest to lowest.
- Move in an S-shape pattern.
- Allow correct contact time.

## DESCRIPTION

CleanLIFE Medical Isopropyl Alcohol Wipes 70% are ready-to-use hospital-grade wipes for disinfecting various surfaces and common touch points. They provide broad antimicrobial effectiveness against bacteria including *Staphylococcus aureus*, *Escherichia coli*, *Pseudomonas aeruginosa*, *Enterococcus hirae*.

## SAFETY DIRECTIONS

This material is hazardous (flammable category 2, eye damage category 2A, specific target organ toxicity category 3) and classified as dangerous goods (class 3).

Keep away from heat/sparks/open flames. No smoking. Keep container tightly closed, ground and bond container and receiving equipment, use explosion-proof electrical, ventilation, lighting and all other equipment, use non-sparking tools. Prevent static discharges. Avoid breathing dust, fumes, gas, mist, vapours or spray. Use only outdoors or in a well-ventilated area. Wear protective gloves/clothing/eye and face protection and suitable respirator.

Keep out of reach of children. Do not use as a personal wipe or on skin. Avoid contact with eyes and wash hands after use. Store in a cool dry place.



Marketed by:  
**Majac Medical Products Pty. Ltd.**  
Tel: 1300 138 578 • Int: +61 7 3265 6355  
Email: [sales@majacmedical.com.au](mailto:sales@majacmedical.com.au)  
[www.majacmedical.com.au](http://www.majacmedical.com.au)



FS 540017