



PLASTIC FREE MULTIPURPOSE DISINFECTANT WIPES

FEATURES

- Alcohol free / chlorine free.
- Proven 99.99% efficacy against tested germs and bacteria.
- Proven to kill the COVID-19 virus in 60 seconds.
- This product is not classified as hazardous or as dangerous goods.

QUANTITY

- Canister 75 wipes and 100 wipes.
- Tub 400 wipes (refills 400 wipes).
- Softpack 40, 80 and 150 wipes.
- Disinfectant spray 1L.

INGREDIENTS

Water, Benzalkonium chloride, Ethylene diamine tetraacetic acid tetrasodium salt, Polyoxyethylenesorbitan monolaurates, Lactic acid.

INSTRUCTIONS

Correct use of the CleanLIFE Medical Multipurpose Disinfectant Wipes helps remove organic matter and improves infection prevention and control. Use a new wipe for each new surface.

Five steps to effective medical cleaning:

- Pull open your tub, canister or softpack.
- Work from cleanest to dirtiest.
- Work from highest to lowest.
- Move in an S-shape pattern.
- Allow correct contact time.



**HARD
SURFACES**



**ALCOHOL
FREE**



**KILLS
COVID-19**



TOUGH



**HAND
SAFE**

DESCRIPTION

CleanLIFE Medical Multipurpose Disinfectant Wipes are ready-to-use hospital-grade wipes for disinfecting various surfaces and common touch point areas. Impregnated with a solution formulated to provide broad antimicrobial effectiveness against bacteria including Staphylococcus aureus, Escherichia coli, Pseudomonas aeruginosa, Enterococcus hirae.

SAFETY DIRECTIONS

Keep out of reach of children. Do not use as a personal wipe or on skin. Avoid contact with eyes and wash hands after use. Store in a cool dry place.



Marketed by:
Majac Medical Products Pty. Ltd.
Tel: 1300 138 578 • Int: +61 7 3265 6355
Email: sales@majacmedical.com.au
www.majacmedical.com.au



FS 540017